

PRODUCT MANUAL



UPLIFT COMMODE

I. INTRODUCTION AND NOTES

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- Prevent unauthorized use, for example, by children.
- Take note of the maximum user weight permitted.

II. PRODUCT DESCRIPTION

Parts of the product



1. Backrest
2. Armrest
3. Seat
4. Leg tube
5. Rubber tip
6. Commode bucket

III. INSTRUCTIONS FOR USE

Assembly

Step 1

Attach backrest tube to the frame, by depressing push buttons and sliding the backrest into the tubes located under the seat.



Step 2

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs. Make sure that push button “snap” correctly through holes and are stable and adjusted the same way.



Step 3

Slide Commode Bucket into guiderails located under the seat.



IV. SPECIFICATIONS

Product Specifications	
Product Dimension	(30"-31") x (23"-24") x (34"-38")
Weight Capacity	300 lbs
Packaging Dimensions	25" x 17" x 22"
Net Weight	16 lbs
Material on product	Aluminum

Warranty

Journey Health & Lifestyle warrants the SoftSecure Uplift Commode frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.

