

Low Cost Pager Reduces Falls And Increases Caregiver Freedom

Use silent or quiet monitor with wireless signal to pager



Reset Button
Press to reset monitor
from alarming

Low Battery Light
Notifies caregiver that
batteries need to be
changed

Mute Call Button
Resident is able to alert
caregiver that they are
in need of assistance
by pressing button on
monitor



Caregiver Key Reset
Optional reset for
caregivers only, with use
of Caregiver Key

Pod Lost Light
Lets caregiver know
that the wireless signal
between the sensor/pad
and monitor have lost
communication

Also works with:



Floor Mats



Seat Belts



Bed Sensor Pads



Wireless Control Monitor

Paging Directly from Fall Monitor

Call button on the front of the monitor can be pressed by the resident if they are in need of assistance. When pressed a wireless signal is sent to the caregiver pager to alert the caregiver that assistance is needed.

Eliminate Bedside Alarms

The monitor can be set to silent to reduce in room alarm noise and reduce startling the resident when alarming. Even though the alarm in the room is set to silent and there is no audible alarm, a wireless signal is sent to the caregivers pager to alert them that the resident is in need of assistance.

On/Off Caregiver Key PATENTED TECHNOLOGY

In optional Caregiver Key Mode, the monitor can only be reset by the caregiver by placing the Caregiver Key over the red dot – eliminating resident tampering. When monitor is not in caregiver key mode, caregiver key will not work.

ITEM	SMART PRODUCT DESCRIPTION
TL-2016	Wireless Fall Monitor (1 year warranty)
TL-2016P	Caregiver Pager with LCD Display for use with TL-2016
PPB-R1	Bed Pad 10"x30", (1 Year Pad Life)
PPC-W1	Chair Pad 10"x15", (1 Year Pad Life)
AC-02	Optional AC Adapter



PRISM HEALTH SERVICES, LLC.

Be Home, Be Safe, Be Happy.

904-503-1042

**Call
Today!**